
Ergotopia® Akupressur Set

Ergotopia® Acupressure Set

Set d'acupression
Set de acupresión
Set di agopressione
Acupressuurset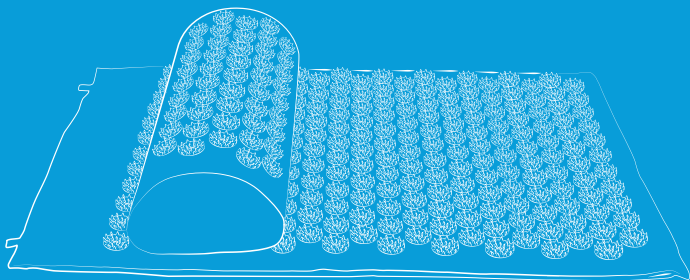

DE Gebrauchsanweisung
EN Instructions for use
FR Mode d'emploi
ES Instrucciones de uso
IT Istruzioni per l'uso
NL Gebruiksaanwijzing

Ergotopia® Akupressur Set

DE

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Ergotopia® Acupressure Set

EN

Please read these instructions carefully before using the acupressure set. Keep the instructions in a safe place and have them handy for use.

Mat: 68 cm x 42 cm, 500 g

Cushion: 35 cm x 16 cm x 10 cm, 150 g

Recommended user weight: 40-120 kg

Recommended duration of use: approx. 10–40 minutes each time

General information

We recommend using the mat especially for soothing relaxation and to escape from everyday stress for a few minutes.

Especially suitable is the use on the back as well as the pillow on the back of the head. If the cushion is initially felt to be too intense, a thin fabric pad or a normal pillow can be used instead. If the feeling of the mat is too intense when in direct contact with the skin, a thin fabric pad can be used.

The mat can also be used for the soles of the feet. To do this, it is best to start by sitting on a comfortable chair and placing the feet either barefoot or with socks on the mat.

CAUTION: Both the thin fabric base and socks can be damaged by using the mat.

Important notes on application

Before use

The Ergotopia® Acupressure Set should not be used, or should be used only after consultation with the attending physician if one or more of the following apply:

- Blood clotting disorders (bruising may occur)
- Open wounds in the area of application
- Heart problems (only in consultation with the attending physician)
- Pregnancy (using the mat should be avoided or previously discussed with the attending physician)
- Acne, eczema, psoriasis, or major skin damage in the area of application (use the mat – if at all – only with an overlay and reduce the duration of application, may only be done in consultation with the attending physician).
- Inflammations in the application area (only in consultation with the attending physician)
- User weight of less than 40 kg or more than 120 kg
- In the case of fibromyalgia and increased sensitivity to pain, the duration of use should initially be limited to a short time. In addition, it is recommended to place a thicker fabric overlay on the mat and to consult with the attending physician.
- Especially in case of an existing whiplash injury, dizziness, and neck pain may increase. In this case, the duration of treatment and the rhythm should be optimized individually in consultation with the doctor and the intensity should be increased slowly.

During use

Be careful when handling the mat (lying down, standing up, unpacking and packing) to protect your hands, eyes, and other sensitive body parts from injury from the spikes.

We do not recommend sleeping (longer) on the mat. Many users report falling asleep while using the Ergotopia® Acupressure Set. Since the duration of use should be 10 to 40 minutes, it is recommended to set an alarm clock if in doubt – lying on the mat for several hours at a time can possibly irritate the skin.

Start with 10 to 15 minutes every 2 to 3 days and slowly increase up to 40 minutes daily (if necessary in consultation with your doctor or therapist). The application is most intense with direct skin contact. However, the intensity of the application can be individually adjusted to your own sensation with the help of a fabric pad (T-shirt, towel, etc.). Before using the Acupressure Set, create a calm and relaxed atmosphere.

In the first few minutes of use, the lying sensation may be somewhat uncomfortable, but after that a very pleasant feeling should develop. A slight feeling of drowsiness may occur temporarily after use. A slight feeling of dizziness or headache may also occur shortly after use.

It is possible for children to use the mat. However, this should definitely be done under adult supervision. Since children's skin is more sensitive, it is advisable to use a cloth pad or a T-shirt.

Material

- The filling material of the mat and the cushion is made of 100% polyurethane.
- The acupressure tips are made of 100 % acrylonitril-butadiene styrene-copolymer (ABS).
- The covers of the cushion and mat are made of 100 % cotton.

Safety notice

The materials used are flammable and do not contain flame-retardant chemicals. Do not expose the mat or cushion to open fire, sparks or other fire-inducing heat sources.

Correct cleaning

Care instructions

The cotton cover is removable and can be washed by hand – beware of the tips! To avoid poking yourself during washing, turn the cover inside out. Please do not expose it to direct sunlight when drying. Not suitable for the dryer.

The carrier bag can be washed at a maximum of 40 °C in the washing machine. Please wash it with similar colors. Not suitable for the dryer.

Disinfection

To disinfect your Acupressure Set, first remove the foam cores from the covers. Then apply or spray a disinfectant suitable for polyurethane foam on the filling material. Air the foam cores for at least 12 hours before reinserting them into the cover.

When washing the covers, use a disinfectant detergent suitable for cotton and ABS.

Recommended: Hand wash at a maximum of 40 °C.

To disinfect the bag as well, wash it in the washing machine with disinfectant detergent as indicated above.

Further helpful information about the Ergotopia® Acupressure Set

Storage and transport conditions

The Acupressure Set should be stored and transported under constant conditions. The optimum temperature is 20 to 24 °C with a humidity of 40 to 60 %.

Scope of delivery

Supplied is 1 x the Ergotopia® Acupressure Set, consisting of a mat and a cushion. The covers are made of cotton fabric with ABS tips, the filling is made of polyurethane foam. A carrying bag and instructions are also included.

Note on the scope of delivery:

If the Acupressure Set arrives differently than pictured or has defects, please contact us immediately at info@ergotopia.de or +49 (0) 6262-8542-977 and refrain from using it for the time being.

Problems during use

In case of problems during use, please contact the responsible dealer or the responsible customer advisor of Ergotopia GmbH and stop using the Acupressure Set for the time being until clarification.

Replacement

The Acupressure Set is not intended for reuse.

Instructions for safe disposal

- Disposal of the acupressure set in household waste.
- Disposal of the packaging material in the yellow garbage can.
- Disposal of paper/cardboard in the blue garbage can.

Recommended service life

3 years

**Enjoy a relaxed and happy
time with your new soothing companion!**

Do you have questions, wishes, or suggestions?
Feel free to contact us at any time!

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